#3000 SPARE PARTS KIT(S)

Important information, read carefully before use

MEDICAL
DEVICE

Manufactured by:

Gynetics Medical Products N.V.
Rembert Dodoensstraat 51
3920 Lommel, Belgium

INTENDED USE

This product is compatible with the #3000 Pipet-ID(s)

REPLACING THE SEALS

The seals need to be replaced every six months or earlier if:

- The Pipette fills with media before the plunger is released.
- The Pipette still fills with media even though the collet was fastened.
- The seals look worn.

REPLACING THE PLUNGER WIRE

The plunger wire needs to be replaced if:

- It is not possible anymore to apply a Pipette over the plunger wire.
- The plunger wire is bent or damaged.
- It is not possible anymore to depress the plunger.

CONTENT

- Plunger needle
- 3 Seals
- Hexagon Key

HOW TO USE

HOW TO REPLACE THE SEALS

- Loosen the collet (**See A**).
- Remove the old seals from the plunger wire, or from the inside of the collet if they are stuck (the blunt end of a used Pipette can be used for this), and discard them.
- Inspect the plunger wire and the inside of the collet, and clean and disinfect, if necessary.
- Slide the seals over the plunger wire in the following order (**See C**):
1st Largest red seal.
2nd Translucent seal, largest hole facing away from the body of the Pipet-ID(s).
3rd Smallest red seal.
- If the seals are applied in any other quantity, order or orientation as described above the Pipet-ID(s) will not function properly.

INSERTING A NEW PLUNGER WIRE

- Loosen the collet (**See A**).
- Take out the three seals (**See C**).
- Loosen the broader end of the Pipet-ID(s) body anti-clockwise until it can move over the plunger (**See D step 1**).
- Turn the thumb rest anti-clockwise to loosen the mechanism from the body (**See D step 2**).
- Carefully loosen the two grub screws by using the hexagon key, supplied with the Pipet-ID(s), until the plunger wire is released (**See D step 3**).
- Take out the plunger wire (**See D step 4**).





- Insert the new plunger wire in the narrow opening in the mechanism and carefully push it in. Make sure that at least 60 mm remains visible.
- Fasten the two grub screws to tightly attach the new plunger wire (**See D step 3**).
- Gently insert the sliding nut back into the body of the Pipet-ID(s) by turning the thumb rest clockwise (**See D step 1 and 2**).
- Turn the thumb rest clockwise until the volume markings on the plunger wire are no longer visible
- Bring the broader end of the Pipet-ID(s) body clockwise back into place (**See D**).
- Turn the thumb rest anti-clockwise until it stops to reset the aspirate volume setting to its maximum of 3µl (**See B**).
- Place back the three seals and the collet (**See C**).
- Test the Pipet-ID(s) to guarantee a smooth movement and repeat steps if necessary.

CAUTION

- The Spare Parts Kit(s) should only be used by qualified personnel.
- Before use, make sure the Pipet-ID(s) is properly cleaned and disinfected by wiping with isopropyl alcohol or similar cleaning agents.
- Check before use the functioning of the #3000 Pipet-ID(s)

TROUBLESHOOTING		
PROBLEM	POTENTIAL CAUSE	SOLUTION
Pipette does not fit	Seals not correctly fitted	Follow instructions 'Replacing the seals'
	Plunger wire damaged	Replace plunger wire
Plunger does not depress	Thumb rest tightened to body	Reset sample delivery to 1- 3 µl
	Plunger wire damaged	Replace plunger wire
	Seals damaged	Replace seals
	Plunger wire loose	Secure plunger wire
Aspiration volume cannot be adjusted	Not correctly maintained	Follow instructions 'Replacing the plunger wire'
	Pipet-ID(s) damaged	Contact Gynetics

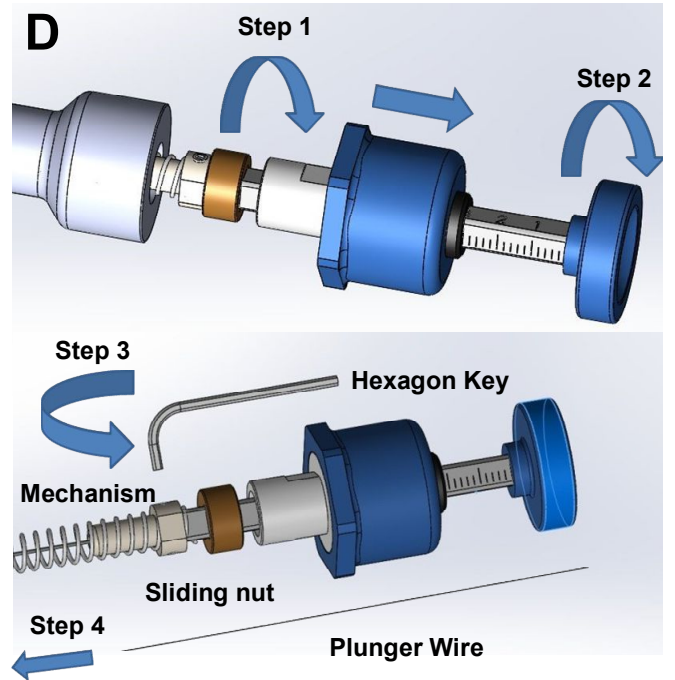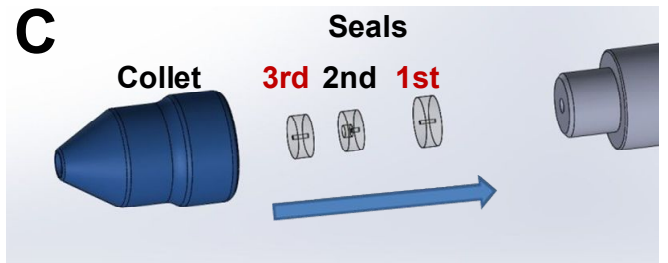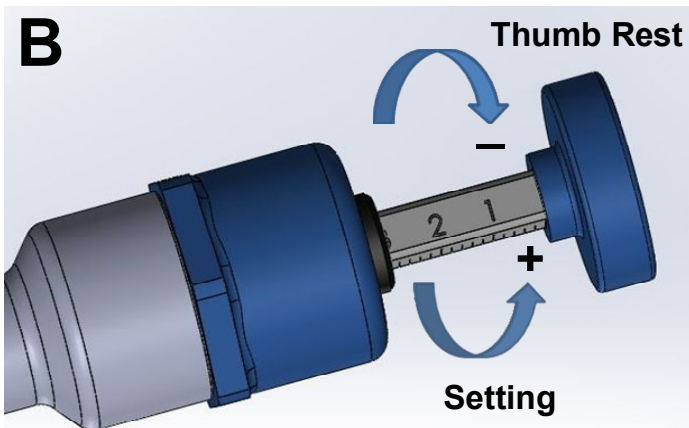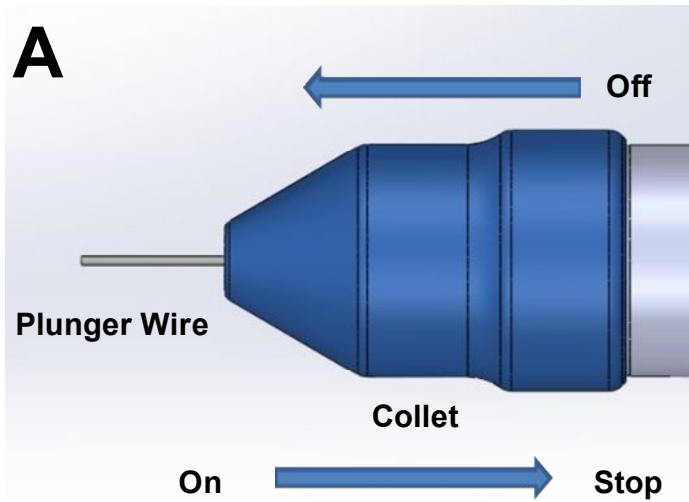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